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(Please read completely before use.)

The easy and comfortable method to rinse your nasal cavities with saline solution to effectively treat symptoms associated with allergies, colds, the flu, or sinus related infections.

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## What is Nasaline®?

Nasaline® is a unique, patented, drug-free nasal rinsing system designed by a team of Swedish ENT specialists. It effectively washes away unwanted mucous, crusty secretions or allergens from the nasal cavities with the simple use of saline solution. After a prolonged stay in dry, dusty, or smoky environments, Nasaline® cleans and soothes the nasal passages, reducing nasal irritation and moistening the mucous membranes. Nasaline® can be used to cleanse and relieve nasal passages affected by:

- Nasal Allergies, Dryness and Hay Fever.
- Sinus Pressure and Nasal Stuffiness
- Nasal Symptoms from Flu, Cold and Sinusitis
- Post-Nasal Drip and Nasal Congestion
- Nasal Irritation from Occupational and House Dust, Fumes, Animal Dander, Grass, Pollen, Smoke, etc.
- Saline Nasal Irrigation, Moisturization and Daily Nasal Hygiene
- For Use After Sinus Surgery and for Sinus Symptoms Suffered by Cystic Fibrosis Patients

## Advantages

- Gentle, comfortable and smooth-flowing saline nasal irrigation
- Silicone tip designed to fit all children's noses to completely seal the nostril opening
- Channel inside the tip is formed in the shape of an hourglass, producing a gentle whirl stream when the saline solution enters the nose
- Syringe feature allows the user to control the flow for optimum comfort and effectiveness
- Excellent for cleansing prior to the use of nasal medications
- Easy to clean to prevent bacterial build-up
- Drug-free and preservative-free; non-sedating
- Can be used as a pH balanced and isotonic solution with one packet, or a hypertonic solution with two packets.
- Suitable for use after sinus surgery or during pregnancy; consult your doctor with concerns
- Portable for travel

## Who can use Nasaline®?

Anyone 2 years or older can use Nasaline®. However, children from ages 2-12 should not use Nasaline®, without parental supervision, unless advised otherwise by your child's doctor. We suggest Nasaline® Junior, a smaller unit, for children ages 2 to 12, also used with parental supervision. We recommend the **NeilMed® Baby NasaKleen®** Nasal Aspirator for newborns and toddlers.

## Dosage

Use two (2) full syringes twice in each nostril per rinsing session. Rinse once or twice daily or as recommended by your doctor

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## Cleaning

Remove the tip, pull out the plunger from the syringe and flush all 3 parts with warm water. The Nasaline® applicator can also be cleaned with a mild dish washing detergent. Dry all 3 parts. Once dry, keep the Nasaline® unit stored with the plunger inserted into the cylinder.

## Three Easy Steps

1. Empty 1 packet (½ teaspoon) of Nasaline® salt into the mixing cup. 
2. Fill the mixing cup to the 8 oz/240 mL line with distilled, micro-filtered (through 0.2 microns), commercially bottled or previously boiled and cooled down water at lukewarm or body temperature. Stir to properly mix the saline solution. Extract the saline solution completely into the Nasaline® applicator. 
3. Stand (or sit) over a bathroom sink or use in the shower. Put the silicone tip against the nose so that it completely seals the nostril. Breathe normally through the mouth (or say aaaah...) and slowly push the saline solution up into the nostril. Refrain from swallowing; when the saline solution reaches the back of the nasal cavities, the palate closes automatically by reflex. The saline fills the nasal cavities and proceeds to pour out the other nostril. Perform this process twice in each nostril. Use a tissue to blow out any remaining saline in your nose. Some of the saline can remain in the nose and trickle out later. Tilting your head forward and turning side to side immediately after flushing can prevent this. 

## ⚠ WARNING

 DO NOT USE TAP WATER UNLESS STERILIZED BY BOILING

- For your safety, do not use tap or faucet water for dissolving the mixture unless it has been previously boiled for five minutes or more (and then cooled down) as boiling sterilizes the water. Other choices are distilled, micro-filtered (through 0.2 micron), reverse osmosis filtered, or commercially bottled water.
- Each device must be used only by one user; do not share this product with others.
- Follow the cleaning protocol after each use of the product. Cleaning instructions can be found in the enclosed brochure.
- Replace this device within three months of your first use.
- Rinse your nose gently.
- Before using this product, you must read detailed warnings in the enclosed brochure or on our website at: [www.neilmed.com/directions](http://www.neilmed.com/directions). Our website will always have up-to-date warnings.
- Store in a cool and dry place.

## ATTENTION

Medical research recommends following the cleaning protocol fully described in the enclosed brochure for Nasaline® after every use and replacing the device after three months.

**Choking Hazard: Small Parts; Keep out of reach of children!**

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